



## Product Family Honeywell Titmus Software

Honeywell Titmus Software is a specialised Vision Screener management software developed for the Honeywell Titmus V4 Vision Screener. A demo version of the software is available for 10 days [Download Software Version 2015.01](#) and [Quick Start Guide](#)



### Product Numbers & Ordering Information

Product Numbers	Details
-----------------	---------



19386

**TITMUS SOFTWARE & CABLE,V**

Software for Titmus VSeries Vision Screeners, model Titmus V4  
Includes cables to connect to vision screener

#### Additional Information

##### Software Order Process:

1. Customer places order with our distributor. Distributor collects information from customer.
  2. Distributor completes the **Software Order Form** and places order with us along with the Software Order Form.
  3. We drop-ship the following to the customer directly
- Cables that connect the PC (software) to the vision screening instrument.
  - Page that shows the webpage to download the software AND activation code.
  - 20 days to use the activation code

### Overview

#### Key Features

##### Subjects

- Manage your facilities subjects (employees) and keep track of their key Occupation Health details.

##### Vision

- Control your vision screener directly from the PC and automatically record your subject's vision test results. Supports Honeywell TITMUSi400 and the new Titmus V4.

##### Integration

- Open interfaces provide accessible integration points for subject and test information to & from other systems; e.g. SAP.

##### Business Intelligence

- Advanced statistics and reporting infrastructure allows you to extract all the information needed for printing, creating .pdf for e-mail, or creating .csv for manipulating the data in Excel.

##### Standards

- Can customize per your requirements or use the generic standards

##### Pass/Fail Viewing

- Can see the pass/fail results for each test on the screen, as you perform the test

##### Corrected / uncorrected vision testing

- Can test a person with and without glasses and compare the results

### Reverse

- Can go back to an individual test within a complete test, if you want to redo the test. Example: You have completed the acuity tests and are now doing the color vision test. You can easily go back to do the acuity test again.

### DEMO VERSION OF SOFTWARE

The **Demo Version** of the software is almost a fully functional version of our software. This is to help our customers review the software before purchase. The only difference between the demo version and the fully functional version is as follows –

- No cable provided with the demo version. Hence you cannot connect a vision screening instrument to the demo version.
- Section of the software that shows the instrument status is unavailable in the demo version, since it is not connected to the vision screening instrument.
- Can set up patients, do a full test, create a test report. But you cannot print a report from the demo version of the software.
- The demo version does NOT need an activation key to try out the software. You can ignore this screen and test the software.

## Specifications

### Memory Requirements

- Minimum 512 MB, 2GB required for large sites (more than 100,000 patients).

### Operating System

- Windows XP or later

### Java

- Oracle Sun Java Runtime 6

### Can be used with Occupational Health Slide Sets only

- (Landolt, Snellen, TNO)

### Software Version

- 2015.01

## Literature & Documents

### VS Software Order Form\_rev M\_2015-03

[http://www.honeywellsafety.com/Supplementary/Documents\\_and\\_Downloads/Eye\\_and\\_Face\\_Protection/4294989809/1033.aspx](http://www.honeywellsafety.com/Supplementary/Documents_and_Downloads/Eye_and_Face_Protection/4294989809/1033.aspx)

### SoftwareQuickStartGuide

[http://www.honeywellsafety.com/Supplementary/Documents\\_and\\_Downloads/Eye\\_and\\_Face\\_Protection/Vision\\_Screening/4294971115/1033.aspx](http://www.honeywellsafety.com/Supplementary/Documents_and_Downloads/Eye_and_Face_Protection/Vision_Screening/4294971115/1033.aspx)

## Additional Information

Download Honeywell Titmus Software Version 2015.01

Download Quick Start Guide

Software Version History

## Technical Support

### Sperian Protection Optical Inc, a Honeywell company

690 HP Way

Chester, VA 23836

Tel: 1-800-446-1802 (US) or +1-804-452-5200 (Intl)

Email: [info@titmus.com](mailto:info@titmus.com)

### Australia/New Zealand

Email: [h1shspamarketing@Honeywell.com](mailto:h1shspamarketing@Honeywell.com)

Ph Aus: 1300 139 166

Ph NZ: 0800 322 200

**RELATED PRODUCTS**

© Honeywell International Inc.